# VOCO MTA vpt

## Application Capsule

### EN Instructions for use MD EU Medical device

#### Product description:

**MTA vpt** is a biocompatible, radiopaque, hydraulic, self-curing cement.

#### Indications:

- Base fill cement for all restorative materials
- Indirect pulp capping
- Direct pulp capping
- Treatment of pulp following vital amputation

#### Contraindications:

**MTA vpt** contains tricalcium aluminate and polyvinylpyrrolidone. **MTA vpt** should therefore not be used in patients with a known hypersensitivity (allergy) to these ingredients.

#### Patient target group:

**MTA vpt** is suitable for use in all patients without any age or gender restrictions.

#### Performance features:

The product's performance features satisfy the requirements of the intended use and the relevant product standards.

#### User:

**MTA vpt** should only be used by a professionally trained dental practitioner.

#### Preparation:

**MTA vpt** is applied as the first layer on the dentine following excavation. Do not use any adhesive/bonding agent or additional liner before applying **MTA vpt**! Ensure that the work area is sufficiently dry. The dentine should not be too moist prior to the application of **MTA vpt**: any standing moisture on the tooth surface, especially from saliva, must be removed. Depending on the clinical procedure employed for caries removal/management, small areas of carious dentine close to the pulp may remain intact in the cavity.

#### Activating the application capsule and mixing the material:

Only extract the application capsule from the tubular bag immediately before use. To use the application capsule, one of the enclosed plungers of liquid first of all has to be inserted with light pressure into the appropriate opening in the capsule body (see fig. 1). Insertion of the plunger must be done without excessive pressure, as its foil seal may otherwise be damaged.

Before further use, the capsule needs to be activated by pressing the liquid from the plunger into the inside of the capsule as fully as possible. To achieve this, the end of the application capsule where the plunger of liquid protrudes is slowly pressed down onto a hard surface (e.g. a table) as far as the plunger will move (see fig. 2). The plunger must only be inserted and the capsule activated directly before use. Then immediately load the activated capsule into a high-frequency mixer with at least 4,000 oscillations per minute and mix for 30 s. Working time and consistency may be influenced by the respective type of capsule mixer and mixing time.

Fig. 1: Preparing the capsule

Fig. 2: Activating the capsule

#### Application:

Remove the mixed application capsule from the mixer immediately and insert it into the holder of an application device (e.g. AC Applicator, see corresponding instructions for use).

Open the outlet by lifting the cannula until you feel it reach the stop (see fig. 3), then immediately extract a small amount of paste onto a mixing pad and discard it.

Fig. 3: Opening the capsule

**MTA vpt** can then (no later than 10 s after completion of mixing) be applied directly into the cavity by compressing the lever of the applicator slowly and evenly.

The cannula can be aligned as required by turning the whole application capsule inside the holder.

Note: Avoid unnecessary waiting times between activation, mixing and application.

To avoid excess pressure in the capsule, the cannula does not completely seal the inside of the capsule. This can lead to slight cement leakage under the cannula during application. This leakage is due to technical reasons and is not critical.

The intraoral working time is at least 3 minutes. For the purposes of caries management, small areas of carious dentine close to the pulp in the cavity must be completely and tightly covered with **MTA vpt**. The mixed paste may appear slightly blue. This colouring disappears after curing, leaving the material white-opaque.

#### Base fill / indirect pulp capping:

Mix the material and cover the base of the cavity. Cover the in situ base fill as described below and place the final restoration.

#### Direct pulp capping:

If necessary, perform haemostasis. Cover the opened pulp and the surrounding dentine with **MTA vpt** and apply the covering layer as described below. Then place the final restoration.

#### Treatment of pulp following vital amputation:

The aim of vital amputation is to preserve parts of the pulp, which is usually inflamed by deep caries or trauma, so as to maintain the vitality of parts of the crown pulp or root pulp, as well as the integrity of the periradicular tissue. In the case of incomplete root growth, this measure can also enable its completion. In addition, root canal treatment on the deciduous tooth can be prevented. In addition to the carious dentine, irreversibly diseased parts of the pulp are also removed.

Healthy pulp tissue is left in place and covered with **MTA vpt**, as are the entire trephine opening and the surrounding dentine. The covering layer is then applied as described below.

#### Covering layer:

Following the application of **MTA vpt**, ensure that the treatment area is clean and dry. Directly after the application, it is carefully covered with a thin layer of flowable, light-curing liner, a flowable, light-curing or self-curing glass ionomer cement or a flowable composite, without exerting pressure, so that the **MTA vpt** is completely covered. Do not perform any conditioning prior to applying the covering layer and do not use an adhesive! Then cure according to the corresponding instructions for use. The remaining cavity is filled with a restorative material as per the corresponding instructions for use. When doing so, pay attention to sufficient material strength to achieve the required mechanical stability.

#### Warnings, precautionary measures:

- **MTA vpt** is highly alkaline – avoid contact with the eyes and mucous membranes. In case of eye contact, rinse well with plenty of water and consult an ophthalmologist.
- Rinse the instruments used immediately with cold water before material residues cure.
- Our information and/or advice do not relieve you of the obligation of checking that the products supplied by us are suitable for their intended purpose.

#### Constituents (in descending order according to content):

Tricalcium silicate, water, calcium tungstate, dicalcium silicate, tricalcium aluminate, polyvinylpyrrolidone, fumed silica

#### Storage instructions and application method:

Store **MTA vpt** dry at 4°C-23°C. Tightly close powder and liquid container immediately after use. Do not use after the expiry date.

#### Disposal:

Dispose of the product in accordance with local regulations.

#### Reporting obligation:

Serious events such as death, temporary or permanent serious deterioration of a patient's, user's or other person's health condition and a serious risk to public health that arise or could have arisen in association with the use of **MTA vpt** must be reported to VOCO GmbH and the responsible authority.

#### Note:

The Summary of Safety and Clinical Performance of **MTA vpt** can be found in the European database on medical devices (EUDAMED – <https://ec.europa.eu/tools/eudamed>). Detailed information can also be found at [www.voco.dental](http://www.voco.dental) / <https://ec.europa.eu/tools/eudamed>.

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VC 60 BB1646 E1 0825 99 © by VOCO