Absorbable Haemostatic Gelatin Sponge

REF **977-0381**

Esonja de gelatina absorbible hemostático
Éponge de gélatine hémostatique résorbable
Resorbierbare blutstillende Gelatineschwamm
Spugna emostatica assorbibile di gelatina
Resorbeerbare gelatinespons hemostatisch
Esonja hemostática absorvível de gelatina

EN

This package insert is not a reference to surgical techniques.

It is designed to assist in using this product.

DESCRIPTION:

Absorbable Gelatin Sponge is sterile malleable, water insoluble sponge intended for haemostatic applications, is made up of highly purified first grade gelatin and sterilized by Gamma irradiation. When implanted in tissues in appropriate amounts, it completely absorbs within < 4 weeks.

INDICATIONS:

In oral and dental surgery, Sponges are an aid in providing haemostasis. Sponge may be used either dry or moistened, depending upon conditions present at operation and preference of the surgeon. Isotonic saline is suitable for use with Sponge.

CONTRAINDICATIONS:

- Do not use sponge in closure of skin incisions because it may interfere with the healing of skin edges.

- Sponge should not be used in patients with known allergies to collagen.

HANDLING INSTRUCTIONS:

- Sponge may be used dry or saturated with physiological salt solution and can be cut into the desired size.
- When applied dry, cut a desired size piece of sponge, roll between the fingers and lightly compress to fit in the cavity or socket. Following the insertion into the cavity or socket, light finger pressure should be applied for one or two minutes.
- When used moistened, cut a desired size piece of sponge and immerse in the saline solution. Then remove from the solution, squeeze thoroughly to remove the air bubbles and replace the sponge in the Saline solution to swell it to its original size. The Sponge piece should be then removed from Saline solution and to be blotted to on sterile gauze to remove the excess fluid. Place the Sponge in the cavity or wound with moderate pressure, using a small cotton gauze or sponge until haemostasis results.
- Removal of the gauze is made easier by wetting it with few drops of sterile saline, to prevent pulling up of the Sponge, which by then should enclose a firm clot.
- For additional applications, fresh pieces should be used and prepared as instructed above. Use only the minimum amount of Sponge necessary to produce haemostasis. Once haemostasis is achieved any excess Gelatin Sponge should be carefully removed.

WARNINGS:

- The over packing of Sponge should be avoided, particularly within bony cavities, since when recovering to its initial volume, Sponge may interfere with normal function of tissues and/or could compress the surrounding tissue, causing necrosis and nerve damage. Although, if Sponge is used in packing, it should be removed after haemostasis.
- Sponge should be used with caution in contaminated wounds. If signs of infection or abscess develop where Sponge has been positioned, reoperation may be necessary in order to remove the infected material and allow drainage.
- Do not re-sterilize. Do not use if the package is opened or damaged. This device is designed and manufactured for single use only.
- Sponge is not intended as a substitute for meticulous surgical technique and the proper application of ligatures or other conventional procedures for haemostasis.

PRECAUTIONS:

- Use only the minimum amount of Sponge needed for haemostasis, holding it at the site until bleeding stops and then removing the excess.
- Sponge should not be used conjunction with methyl methacrylate adhesives.
- Sponge should not be used in conjunction with autologous blood salvage circuits.
- Sponge is not recommended for the primary treatment of coagulation disorders.
- Sponge should not be used in the presence of infection.
- Users should be familiar with surgical procedures and techniques involving gelatin sponge before employing Sponge.
- Sponge sterile sponge is packed in blisters/envelopes sterile barrier which guarantee sterility.
- Once the package is opened, contents are subject to contamination. It is recommended that Sponge should be used as soon as the package is opened and unused contents discarded. Open the blister/envelope pack by pulling its two loose ends away from each other –in such a way that the Sponge falls out undamaged, on to a sterile surface.
- Discard any unused Sponge remaining. Dispose of contaminated devices and packaging materials utilizing standard hospital procedures and universal precautions for bio hazardous waste.

ADVERSE REACTIONS:

- Foreign body reactions, encapsulation of fluid and haematoma have been reported.

STORAGE:

The product should be stored in its original packaging and the outer package must therefore be closed immediately after use. The box should be stored in a clean, dry room at a temperature not more than 30°C.

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AEGIS LIFESCIENCES PVT. LTD.
OBELIS S.A